FertiCult™ Flushing medium

FertiCult™ Flushing medium

Cell culture medium for washing of human ova, spermatozoa and embryos, for swim-up of spermatozoa, sperm injection in ICSI, intra uterine insemination and embryo transfer

STERILE A
 Documento riferimento: FP09 I08 R01 C.6
 Update: 03.12.2019

USED ABBREVIATIONS

ICSI Intracytoplasmic Sperm Injection
 IUI Intra Uterine Insemination

INDICATIONS FOR USE

FertiCult™ Flushing medium is intended for in vitro procedures involving human gametes (sperm and oocytes), including washing of gametes, sperm swim-up procedures, intra-uterine insemination (IUI) of the spermatozoa and sperm injection during intracytoplasmic sperm injection (ICSI). FertiCult™ Flushing medium can also be used for human embryo washing and holding, and for embryo transfer (ET) in the uterine cavity.
For professional use only.

GENERAL INFORMATION

The medium is complete and needs no further additives. The medium contains HEPES, no CO₂ incubation is required. FertiCult™ Flushing medium does not contain heparin, thus for oocyte pick-up, use FertiCult™ Aspiration medium.

COMPOSITION

FertiCult™ Flushing medium is a ready-to-use HEPES-buffered medium which also contains bicarbonate, physiological glucose, lactate and human serum albumin (4g/l). The product is also available with phenol red and/or gentamicin (10mg/l).

MATERIALS INCLUDED

Product code: FLUSH020
 » 5x 20ml FertiCult™ Flushing medium
 Product code: FLUSH22.8
 » 1x 22,8ml FertiCult™ Flushing medium
 Product code: FLUSH050
 » 5x 50ml FertiCult™ Flushing medium
 Product code: FLUSH100
 » 3x 100ml FertiCult™ Flushing medium
 Product code: FLUSH_002_PHR
 » 1x 2ml FertiCult™ Flushing medium with phenol red
 Product code: FLUSH020PHR
 » 5x 20ml FertiCult™ Flushing medium with phenol red
 Product code: FLUSH050PHR
 » 5x 50ml FertiCult™ Flushing medium with phenol red
 Product code: FLUSH100PHR
 » 3x 100ml FertiCult™ Flushing medium with phenol red
 Product code: FLUSH500PHR
 » 1x 500ml FertiCult™ Flushing medium with phenol red
 Product code: FLUSH020PRG
 » 5x 20ml FertiCult™ Flushing medium with Phenol Red and Gentamicin
 Product code: FLUSH050PRG
 » 5x 50ml FertiCult™ Flushing medium with Phenol Red and Gentamicin
 Product code: FLUSH100PHR_G
 » 3x 100ml FertiCult™ Flushing medium with Phenol Red and Gentamicin
 Product code: FLUSH500PHR_G
 » 1x 500ml FertiCult™ Flushing medium with Phenol Red and Gentamicin

Product code: WASH005
 » 1x 5ml Washing/Insemination medium
 Product code: WASH020
 » 1x 20ml Washing/Insemination medium
 Product code: WASH100G
 » 1x 100ml Washing/Insemination medium with Gentamicin

MATERIALS NEEDED BUT NOT PROVIDED

- » Incubator at 37°C (No CO₂)
- » Petri dishes
- » Microscope
- » Test tubes
- » LAF bench (ISO Class 5)
- » Syringe (e.g. 1ml Plastipack)
- » Catheter

PRODUCT SPECIFICATIONS

- » Chemical composition
- » pH release criteria: 7.30-7.60
- » pH stability criteria: 7.30-7.90
- » Osmolality: 270-290 mOsm/kg
- » Sterility test by the current Ph. Eur. 2.6.1 / USP <71>: No growth
- » Endotoxin test by Limulus Amebocyte lysate (LAL) methodology (USP <85>): < 0.25 EU/ml
- » 1-Cell Mouse Embryo Assay (blastocysts after 96h): ≥ 80% after 60min of exposure (zygote stage)
- » Use of Ph Eur or USP grade products if applicable
- » Certificate of analysis and MSDS are available upon request

SYMBOLS AS DEFINED IN ISO 15223-1:2016
 Simbolo definito en la norma ISO 15223-1:2016
Simbolo conforme definido na norma ISO 15223-1:2016
 Symbolen volgens ISO 15223-1:2016
 Σύμβολα όπως ορίζονται εν ISO 15223-1:2016

WARNINGS BEFORE USE

- » Do not use the product if:
 - it becomes discoloured (if medium contains phenol red), cloudy or shows any evidence of microbial contamination
 - seal of the container is opened or defect when the product is delivered
 - expiry date has been exceeded
- » Do not freeze before use
- » Do not re-sterilize after opening
- » Products that include gentamicin should not be used on a patient that has a known allergy to gentamicin or similar antibiotics.

STORAGE INSTRUCTIONS

- » Store FertiCult™ Flushing medium without gentamicin between 2-8°C.
- » Store FertiCult™ Flushing medium with gentamicin between 2-8°C.
- » Once opened store all products between 2-8°C.
- » Keep away from sunlight.
- » The products can be used up to 7 days after opening, when sterile conditions are maintained and the products are stored at 2-8°C.
- » Stable after transport (max. 5 days) at elevated temperature (≤ 37°C).

PRECAUTIONS

- » Aseptic technique should be used to avoid possible contamination, even when FertiCult™ Flushing medium contains gentamicin.
- » All blood products should be treated as potentially infectious. Source material used to manufacture this product was tested and found non-reactive for HbsAg and negative for Anti-HIV-1/2, HIV-1, HBV, and HCV. Furthermore, source material has been tested for parvovirus B19 and found to be non-elevated. No knock test methods can offer assurances that products derived from human blood will not transmit infectious agents.
- » Standard measures to prevent infections resulting from use of medicinal products prepared from human blood or plasma include selection of donors, screening of individual donations and plasma pools for specific markers of infection and the inclusion of effective manufacturing steps for the inactivation/removal of viruses. Despite this, when medicinal products prepared from human blood or plasma are administered, the possibility of transmitting infective agents cannot be totally excluded. This also applies to unknown or emerging viruses and other pathogens. There are no reports of proven virus transmissions with albumin manufactured to European Pharmacopoeia specifications by established processes.
- » Always handle all specimens as if capable of transmitting HIV or hepatitis.
- » Always wear protective clothing when handling specimens.

METHOD

As this medium does contain HEPES, the medium should not be placed in a CO₂ incubator prior to use. Pre-incubation in incubator (without CO₂) or heating plate is required to ensure that the temperature of the medium is 37°C at use. Make sure that the lid is closed when bottles are placed in an incubator with CO₂ as this will lower the pH to 6.90-6.95. Each laboratory should consult its own validated procedures, optimized for its individual medical program. When performing IUI or ET, follow the instructions of the specific catheters used.

Washing of spermatozoa (suggested procedure)

The washing of spermatozoa can be done at room temperature or at 37°C.

1. Add 5ml FertiCult™ Flushing medium to the native semen sample and mix. Centrifuge for 15 minutes at approximately 300g.
2. Remove supernatant and leave about 0.5ml of semen in the centrifuge tube.
3. Add 5ml FertiCult™ Flushing medium to the test tube. Mix the solution gently until the pellet is completely dissolved.
4. Centrifuge again for 10 minutes at 300g.
5. Perform a swim-up.

Swim-up procedure (according to WHO, 2010)

1. Gently layer 1.5ml FertiCult™ Flushing medium over 1ml of washed semen in a conical based centrifuge tube.
2. Incline the tube at an angle of 45° and incubate for 1 hour at 37°C.
3. Gently turn to the upright position and remove the uppermost 1ml.
4. Dilute this aliquot of motile cells with 8 volumes FertiCult™ Flushing medium. Centrifuge for 15 minutes at 300g, remove supernatant and finally resuspended in 0.5ml of FertiCult™ Flushing medium.

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Produktcode: FLUSH020
 » 5x 20ml FertiCult™ Flushing medium
 Produktcode: FLUSH22.8
 » 1x 22,8ml FertiCult™ Flushing medium
 Produktcode: FLUSH050
 » 5x 50ml FertiCult™ Flushing medium
 Produktcode: FLUSH100
 » 3x 100ml FertiCult™ Flushing medium
 Produktcode: FLUSH_002_PHR
 » 1x 2ml FertiCult™ Flushing medium with phenol red
 Produktcode: FLUSH020PHR
 » 5x 20ml FertiCult™ Flushing medium with phenol red
 Produktcode: FLUSH050PHR
 » 5x 50ml FertiCult™ Flushing medium with phenol red
 Produktcode: FLUSH100PHR
 » 3x 100ml FertiCult™ Flushing medium with phenol red
 Produktcode: FLUSH500PHR
 » 1x 500ml FertiCult™ Flushing medium with phenol red and Gentamicin
 Produktcode: FLUSH020PRG
 » 5x 20ml FertiCult™ Flushing medium with Phenol Red and Gentamicin
 Produktcode: FLUSH050PRG
 » 5x 50ml FertiCult™ Flushing medium with Phenol Red and Gentamicin
 Produktcode: FLUSH100PHR_G
 » 3x 100ml FertiCult™ Flushing medium with Phenol Red and Gentamicin
 Produktcode: FLUSH500PHR_G
 » 1x 500ml FertiCult™ Flushing medium with Phenol Red and Gentamicin

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